

# Surshield® Safety Winged Infusion and Blood Collection Sets



***Help shield your staff against accidental needle-sticks with Surshield®***

**Terumo® Surshield® Safety Winged Needle Sets provide:**


- One-handed\*, easy-to-activate safety device
- Minimal steps for activation
- No change in needle insertion technique required
- Confirmation that needle is securely locked
- Minimize the risk of needlestick injuries

\*Note: OSHA bloodborne pathogen standard 1910.1030(d)(2)(vii)(B)


Surshield® Safety Winged Needle Sets have the soft, flexible, interlocking wings that readily conform to body's contour. The needles are bonded to the wings at just under a 3° downward angle, for less risk of vessel damage or occlusion of the lumen. The coil-resistant tubing reduces recoil during use. Surshield® Safety Needle sets are DEHP free and have no components made of natural rubber latex.

# Surshield® Safety Winged Infusion and Blood Collection Sets


## How to Activate

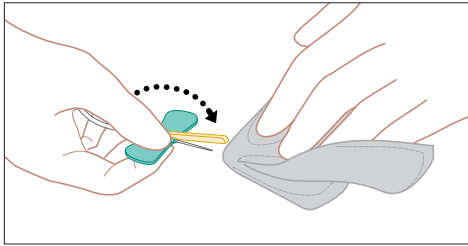

1. Flip safety shield toward needle. Remove the needle and apply digital pressure to the site.

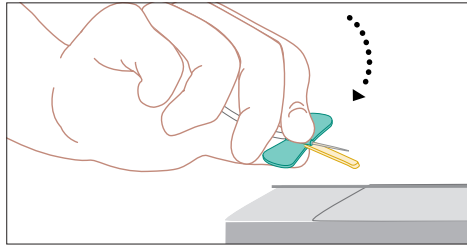


2. With the wing and shield between your thumb and index finger, press the safety shield against a hard surface such as a bedside table until an audible click is heard.

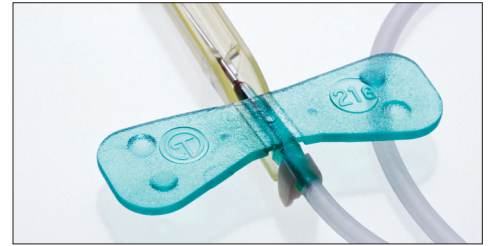

3. Visually confirm activation of the safety feature.

Based on IFU Rev. October 2008.

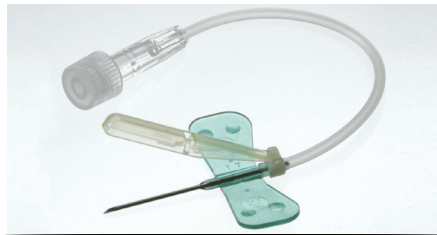

### Surshield® Safety Winged Blood Collection Sets (with attached multi-sample luer adapter)

Packed: 50/box, 10 boxes/case (500)

Product Code	Description
MN*SVS21B30	21G x 3/4" Needle and 12" Tubing
MN*SVS23B30	23G x 3/4" Needle and 12" Tubing
MN*SVS25B30	25G x 3/4" Needle and 12" Tubing
MN*SVS21B18	21G x 3/4" Needle and 7" Tubing
MN*SVS23B18	23G x 3/4" Needle and 7" Tubing

### Surshield® Safety Winged Infusion Sets (without multi-sample luer adapter)

Packed: 50/box, 10 boxes/case (500)

Product Code	Description
SV*S19BL	19G x 3/4" Needle and 12" Tubing
SV*S21BL	21G x 3/4" Needle and 12" Tubing
SV*S23BL	23G x 3/4" Needle and 12" Tubing
SV*S25BL	25G x 3/4" Needle and 12" Tubing
SV*S19BLS	19G x 3/4" Needle and 3 1/2" Tubing
SV*S21BLS	21G x 3/4" Needle and 3 1/2" Tubing
SV*S23BLS	23G x 3/4" Needle and 3 1/2" Tubing
SV*S25BLS	25G x 3/4" Needle and 3 1/2" Tubing

### Multi-Sample Luer Adapter

Packed: 100/box, 30 boxes/case (3,000)

Product Code	Description
MN*2000T	VENOJECT® Multi-Sample Luer Adapter (male). For use with Standard Tube Holder and SampLok® Tube Holder.



### Venoject® II Tube Holder

Packed: 50/bag, 20 bags/case (1,000)

Product Code	Description
P-1316R	VENOJECT® II (Standard) Tube Holder. Use with the multi-sample luer adapter and Diversion Blood Sampling Arm® and Blood Sampling Arm® blood bags to collect samples.

[TerumoTMP.com/SurShield](http://TerumoTMP.com/SurShield)



SURSHIELD, TERUMO and TERUMO are trademarks owned by Terumo Corporation, Tokyo, Japan, and they are registered with the U.S. Patent & Trademark Office. TMP-0199-04042016



TERUMO MEDICAL PRODUCTS