




Surshield® Safety Winged Blood Collection Sets Features and Benefits

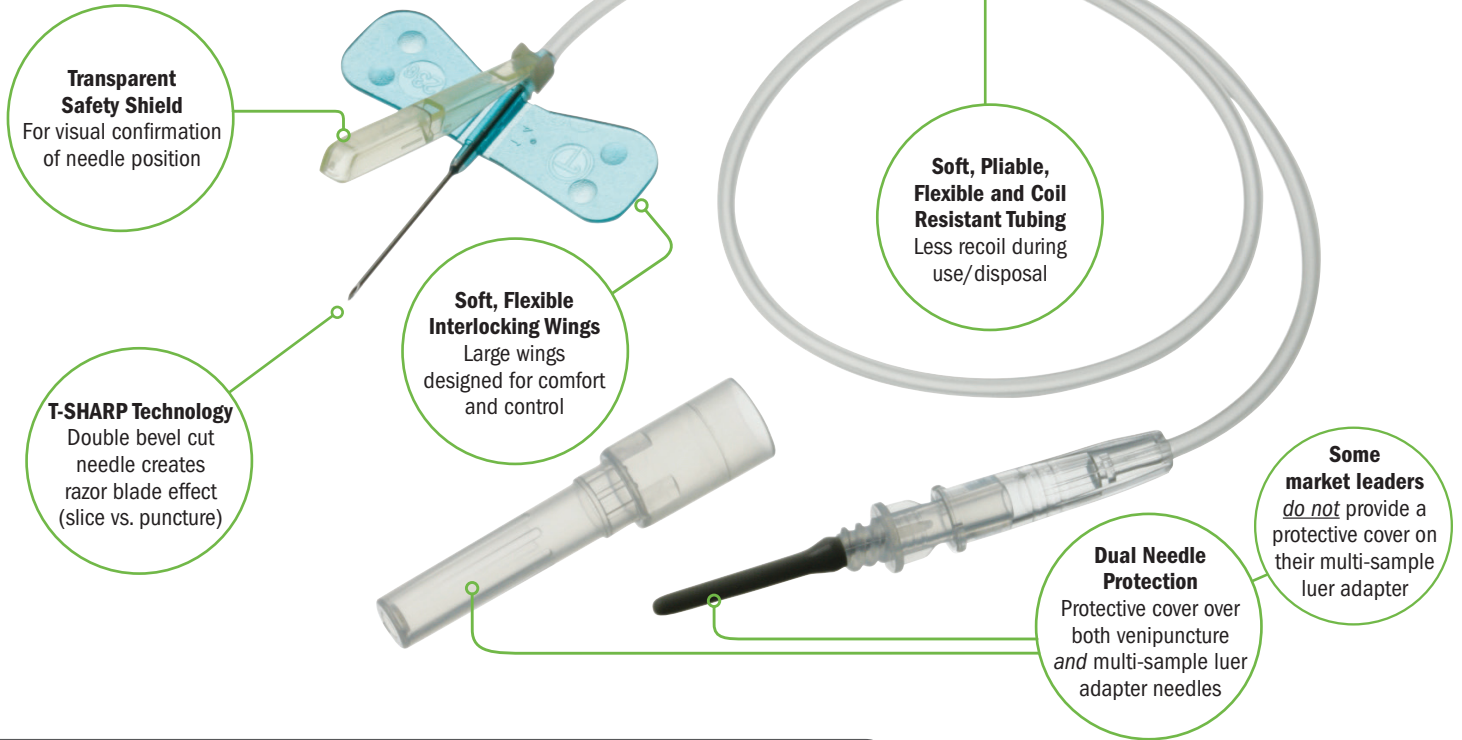


TWO MODES of One-handed Safety Activation


Most Safety Winged Butterflies Are Designed To Be Activated One Way, and Require a Second Hand. Surshield® Provides Two Different One-Handed Activation Options:

THE SHIELD

- Shield cover can be turned 180° on the hinge
- Shield cover is designed to allow the user's finger to remain behind the needle point, minimizing risk of needle stick
- Shield cover is transparent for easy visual confirmation of needle placement
- Safety shield positioned top of wing - indicates bevel up

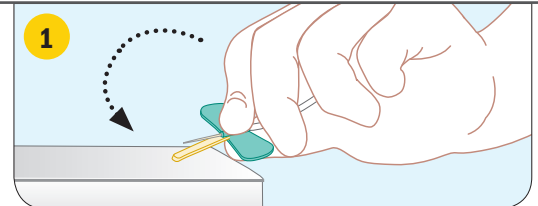

Hard surface - press the safety shield against a hard surface. An audible click signals the shield is in the locked position.

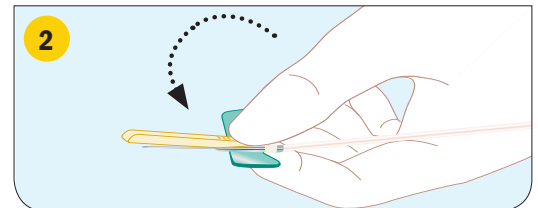

Thumb - use the thumb and forefinger to apply digital pressure to activate. An audible click signals the shield is in the locked position.

RX ONLY. The advertisement is directed to physicians only, and not to consumers. Refer to product labels and packaging insert for complete warnings, precautions, potential complications, and instructions for use. Products may not have regulatory approval in all countries. Please contact your local sales representative if you have questions about the availability of products in your area.

Refer to the product labels and package insert for complete warnings, precautions, potential complications, and instructions for use.

SURSHIELD, TERUMO and TERUMO are trademarks owned by Terumo Corporation, Tokyo, Japan, and they are registered with the U.S. Patent & Trademark Office. PM-03687

